# Vcheck

## Control Canine Tnl

### CONTROL OF CANINE TROPONIN I

For veterinary use only

## INTENDED USE

It is important to perform control tests with more than one level of control materials to assure your system is working properly. Vcheck Control Canine Tnl is intended for use as quality control materials to check the performance of Vcheck Canine Tnl test kit and Vcheck Analyzers.

## PRINCIPLE

Two levels of control help to monitor the performance of Vcheck Canine Tnl test kit by checking whether the measurement value with control and Vcheck Canine Tnl test kit is within the control range. The use of Vcheck Control Canine Tnl materials is indicated as an objective assessment of the precision of methods and techniques in use.

## MATERIALS PROVIDED

Reagent	5 Tests/Kit
① Low concentration (gray pouch) disposable tablet pipette (blue bulb)	5
② High concentration (orange pouch) disposable tablet pipette (orange bulb)	5
③ Instructions for use	1

## MATERIALS REQUIRED, BUT NOT PROVIDED

- BIONOTE Vcheck Analyzer
- Vcheck Canine Tnl test kit

## STORAGE AND STABILITY

- Store the control at 2~30 °C. **DO NOT FREEZE.**
- Do not store the control in direct sunlight.
- The control is stable until the expiry date that is marked on the package label.

Reagent	Open status	Storage	Stability	Note
Control Low	Unopened	2~30 °C, Sealed	24 months	Finished product
	Opened	Do not store	-	Use directly
Control High	Unopened	2~30 °C, Sealed	24 months	Finished product
	Opened	Do not store	-	Use directly

## PRECAUTIONS


- This reagent needs to be stored at 2~30 °C. If refrigerated, allow all kit components to reach room temperature (15~30 °C) for at least 30 minutes prior to testing.
- Tablet pipette is sensitive to humidity and heat. Perform the test immediately after removing the test device from the aluminum foil pouch.
- Fully dissolve the control with the assay diluent. Strictly follow the instructions for use including the required reagent volume and testing time to avoid erroneous results.
- Do not use the reagent if the expiry date has passed.
- Decontaminate and dispose of all samples, used kits and potentially contaminated materials safely in accordance with national and local regulations.
- It is recommended to use the BIONOTE Vcheck Analyzer at 15~30 °C.
- Perform the test immediately after dissolving the control material.
- The dissolved control material is for one time use only. Do not store/reuse the control material that has been dissolved.
- Do not use the control if the pouch is damaged or the seal is broken.
- Control test should be performed only with the appropriate test kit.

## TEST PROCEDURE


If Vcheck Canine Tnl test kit and Vcheck Control Canine Tnl are stored at 2~8 °C, allow kit components of Vcheck Canine Tnl test kit and Vcheck Control Canine Tnl to room temperature (15~30 °C) at least 30 minutes before starting the test.

## [Coding]

- Turn on V200 Analyzer and select "QC".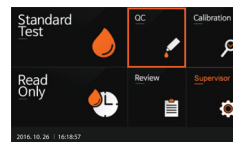

- Enter Control ID written on the control pouch and select "Direct" button.

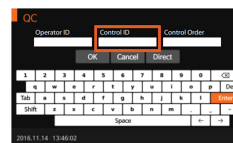
\* Please note that Control Low and High have different Control ID.

- Remove the test device from the aluminum foil pouch of Vcheck Canine Tnl test kit and insert the device into Vcheck Analyzer so that the analyzer can read the barcode first.

## [Dilution of Vcheck Control Canine Tnl & Measurement]

- Remove Vcheck Control Canine Tnl - Low tablet pipette (blue bulb) from gray pouch and mix well with the assay diluent tube from the Vcheck Canine Tnl test kit by gently squeezing the rubber bulb until the white tablet is completely dissolved.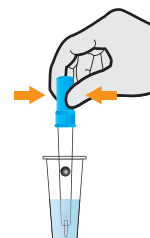


\* Aspirate only half of the diluent to minimize foam.

\* Use a Blue disposable tablet pipette in the Vcheck Control Canine Tnl pouch


\* For testing Vcheck Control Canine Tnl high concentration, use Vcheck Control - High tablet pipette (orange bulb) from orange pouch and follow the test procedure.

- After the white tablet is completely dissolved, add all diluent mixture into the sample hole of the test device using the disposable tablet pipette and press [START] button immediately to initiate testing.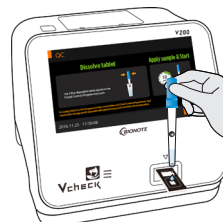

\* Caution: If the time to press [START] button is delayed, it may affect the test result.

- The V200 Analyzer will display the test result on the screen after 10 minutes.
- Remove the test device.

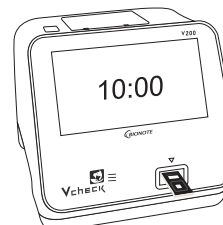## INTERPRETATION OF THE RESULT

- [Passed] or [Failed] result with the concentration of the Vcheck Control Canine Tnl will be on the screen.
- If "Passed" appears on the display, it means the result value is within the reference range and the Vcheck Canine Tnl test kit is valid.
- If "Failed" appears on the display, it means the result value exceeds reference range and Vcheck Canine Tnl test kit is invalid.

Manufactured by BIONOTE, Inc.  
22, Samsung 1-ro 4-gil, Hwaseong-si, Gyeonggi-do 18449, Republic of Korea  
TEL: 82-31-211-0516 | FAX: 82-31-8003-0618 | www.bionote.co.kr

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