# Vcheck Control

## Canine NT-proBNP

CONTROL OF CANINE NT-proBNP

For veterinary use only

### INTENDED USE

It is important to perform control tests with more than one level of control materials to assure your system is working properly. Vcheck Control Canine NT-proBNP is intended for use as quality control materials to check the performance of Vcheck Canine NT-proBNP test kit and Vcheck Analyzers.

### PRINCIPLE

Two levels of control help to monitor the performance of Vcheck Canine NT-proBNP test kit by checking whether the measurement value with control and Vcheck Canine NT-proBNP test kit is within the control range. The use of Vcheck Control Canine NT-proBNP materials is indicated as an objective assessment of the precision of methods and techniques in use.

### MATERIALS PROVIDED

| Reagent  | 5 Tests/Kit |
|--|-------------|
| ① Low concentration (gray pouch)<br>disposable tablet pipette (blue bulb)      | 5           |
| ② High concentration (orange pouch)<br>disposable tablet pipette (orange bulb) | 5           |
| ③ Instructions for use   | 1           |

### MATERIALS REQUIRED, BUT NOT PROVIDED

1. BIONOTE Vcheck Analyzer
2. Vcheck Canine NT-proBNP test kit
3. 100 µl pipette

### STORAGE AND STABILITY

1. Store the control at 2~30°C. **DO NOT FREEZE.**
2. Do not store the control in direct sunlight.
3. The control is stable until the expiry date that is marked on the package label.

### PRECAUTIONS


1. This reagent needs to be stored at 2~30°C. If refrigerated, allow all kit components to reach room temperature (15~30°C) for at least 30 minutes prior to testing.
2. Exposure to humidity and heat may lead to deformation of the control and cause instability or deterioration of the test result. Remove the control from the foil pouch right before testing and perform the test within 10 minutes from the removal.
3. Fully dissolve the control with the assay diluent. Strictly follow the instructions for use including the required reagent volume and testing time to avoid erroneous results.
4. Do not use the reagent if the expiry date has passed.
5. Discard all used test materials and specimens after processing autoclave for a minimum of 15 minutes at 121°C.
6. It is recommended to use the BIONOTE Vcheck Analyzer at 15~30°C.
7. Perform the test immediately after dissolving the control material.
8. The dissolved control material is for one time use only. Do not store/reuse the control material that has been dissolved.
9. Do not use the control if the pouch is damaged or the seal is broken.

### TEST PROCEDURE


If Vcheck Canine NT-proBNP test kit and Vcheck Control Canine NT-proBNP are stored at 2~8°C, allow kit components of Vcheck Canine NT-proBNP test kit and Vcheck Control Canine NT-proBNP to room temperature (15~30°C) at least 30 minutes before starting the test.

### [Coding]

1. Turn on V200 Analyzer and select "QC".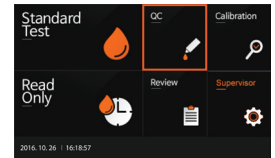

2. Enter Control ID written on the control pouch and select Direct button.

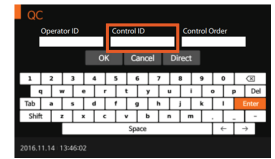
\* Please note that Control Low and High have different Control ID.

3. Remove the test device from the foil pouch of Vcheck Canine NT-proBNP test kit. And insert the device into Vcheck analyzer so that the analyzer can read the barcode first.

### [Dilution of Vcheck Control Canine NT-proBNP & Measurement]

1. Remove Vcheck Control Canine NT-proBNP - Low tablet pipette from gray pouch and mix well with the assay diluent bottle from the Vcheck Canine NT-proBNP test kit by gently squeezing the rubber bulb until the white tablet is completely dissolved.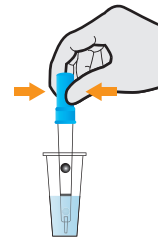


\* Aspirate only half of the diluent to minimize foam.

\* Use a Blue disposable tablet pipette in the Vcheck Control Canine NT-proBNP pouch


\* For testing Vcheck Control Canine NT-proBNP high concentration, use Vcheck Control Canine NT-proBNP - high concentration (orange pouch) disposable tablet pipette (orange bulb) and follow the test procedure.

2. After the tablet is completely dissolved, draw 100 µl of diluent mixture using a 100 µl pipette and add it into the sample hole of test device. Then, press [START] button immediately to initiate testing.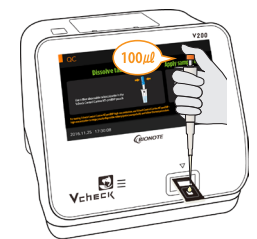

\* Caution: If the time to press [START] button is delayed, it may affect the test result.

3. The V200 analyzer will display the test result on the screen after 15 minutes.
4. Remove the test device.

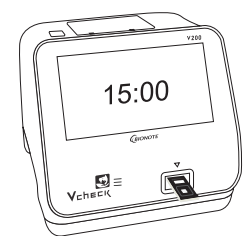### INTERPRETATION OF THE RESULT

1. [Passed] or [Failed] result with the concentration of the Vcheck Control Canine NT-proBNP will be on the screen.
2. If "Passed" appears on the display, it means the result value is within the reference range and the Vcheck Canine NT-proBNP test kit is valid.
3. If "Failed" appears on the display, it means the result value exceeds reference range and Vcheck Canine NT-proBNP test kit is invalid.

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