For veterinary use only

INTENDED USE

The Vcheck CHW Ag is an *in vitro* diagnostic test kit for the qualitative detection of canine *Dirofilaria immitis* antigen in canine serum, plasma, or whole blood. The Vcheck CHW Ag is designed to be used only by veterinarians.

PRINCIPLE

The Vcheck CHW Ag Test Kit is a chromatographic immunoassay for the qualitative detection of canine *Dirofilaria immitis* antigen in canine serum, plasma, or whole blood. Test line and control line in the result window are not visible before applying any samples. The control line is a reference line which indicates the test is performing properly. The control line has to appear every time when the test has performed. If the canine *Dirofilaria immitis* antigens are present in sample, a purple test line would appear in the result window. The highly selective antibodies to canine *Dirofilaria immitis* antigens are used as a capture and detector in the assay. These antibodies are capable of detecting canine *Dirofilaria immitis* antigen in sample with high accuracy. The Vcheck Analyzer reads the presence of canine *Dirofilaria immitis* antigen in canine serum, plasma or whole blood.

MATERIALS PROVIDED

Reagent	10 Tests/Kit
① Vcheck CHW Ag Test device	10
② Assay diluent bottle	1
③ Anticoagulant tube	10
④ Pipette tip	10
⑤ Instructions for use	1

MATERIALS REQUIRED, BUT NOT PROVIDED

1. Timer
2. BIONOTE Vcheck Analyzer
3. 25 µl pipette
4. 50 µl pipette

STORAGE AND STABILITY

1. Store the test kit at 2~30°C. **DO NOT FREEZE.**
2. Do not store the test kit in the direct sunlight.
3. The test kit is stable within the expiration date that marked on the package label.

PRECAUTIONS


1. The test kit is for canine use only. Do not use for other animals.
2. The test device is sensitive to humidity as well as heat. Perform the test immediately after removing the test device from the foil pouch.
3. Do not reuse test components.
4. Apply the sample using pipette vertically.
5. Do not touch the membrane in the result window of test device.
6. Do not use the test kit beyond the stated expiration date marked on the package label.
7. Do not use the test kit if the pouch is damaged or the seal is broken.
8. Do not mix components from different lot numbers because the components in this kit have been quality control tested as standard batch unit.
9. All samples should be handled as being potentially infectious. Wear protective gloves while handling samples. Wash hands thoroughly afterwards.
10. Decontaminate and dispose of all samples, reaction kits and potentially contaminated materials safely in accordance with national and local regulations.

COLLECTION AND PREPARATION OF SAMPLE

1. The test should be performed using serum, plasma, or whole blood (anticoagulated).
2. If the test kit and specimens are refrigerated, they should be brought to room temperature prior to use.

[Whole blood]

- Blood should be collected with a disposable syringe and added to a tube containing anticoagulant (Heparin, EDTA or Citrate). For the user's convenience, Vcheck CHW Ag Test Kit provides anticoagulant tubes (Max. vol. 1.5ml). Please follow the directions below.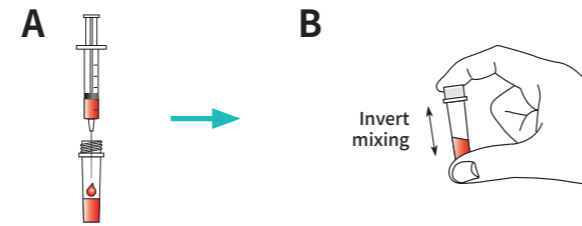


- A. Put the collected blood sample into the anticoagulant tube.
- B. Close the cap on the anticoagulant tube and invert the tube five times to mix the blood and the EDTA.

- Collected blood should be tested immediately or within 4 hours at room temperature. It must be stored at 2~8°C for 24 hours.
- * Note : Blood samples should not be frozen prior to testing.
- Severely hemolyzed blood samples may affect the result.


[Plasma]

- Blood should be collected with a disposable syringe and added to a tube containing anticoagulant (Heparin, EDTA or Citrate), and then separate plasma by centrifugation.
- Plasma must be stored at 2~8°C for up to 3 days. For longer storage, freeze at below -20°C.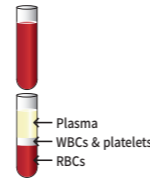

[Serum]

- Blood should be collected with a disposable syringe and added to a serum collection tube (no anticoagulant).
- Collected blood should be left at room temperature for 30 minutes to coagulate, and then separate serum by centrifugation.
- Serum must be stored at 2~8°C for up to 3 days. For longer storage, freeze at or below -20°C.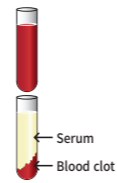


TEST PROCEDURE


All reagents and samples must be at room temperature (15~30°C) before use.

“Standard Test” mode

1. Turn on V200 Analyzer and select “Standard Test” on the analyzer's screen.
2. Remove the test device from the foil pouch.
3. Once the “Insert Device” is displayed in the screen, insert the test device into the V200 Analyzer.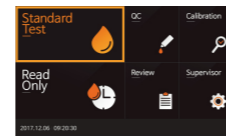

4. Use a 50 µl or 25 µl pipette to draw of whole blood 50 µl or serum(plasma) 25 µl and add the sample into the sample hole.

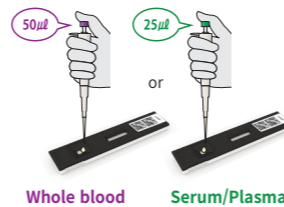5. Add two (2) drops of the assay diluent into the sample hole of the test device. And then, press the [START] button.



- * Caution: If the time to press [START] button is delayed, it may affect the test result.

6. Read the result on the display after 10 minutes.
7. The V200 analyzer will automatically display the test result on the screen.
8. Remove the test device.


“Read Only” mode

1. Remove the test device from the foil pouch and place it on a flat and dry surface.
2. Use a 50 µl or 25 µl pipette to draw of whole blood 50 µl or serum(plasma) 25 µl and add the sample into the sample hole.
3. Add two (2) drops of the assay diluent into the sample hole of the test device. And then, leave the test device for 10 minutes.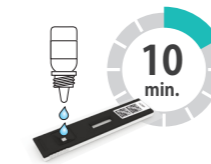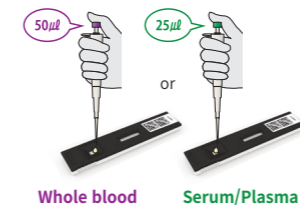

- * Caution : Note that the test device should not be left more than 13 minutes.

4. Turn on V200 Analyzer and select “Read Only” on the analyzer's screen.

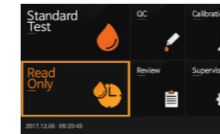5. Insert the test device to the V200 Analyzer.
6. The analyzer will automatically display the test result on the screen.
7. Remove the test device.

INTERPRETATION OF THE RESULT

[V200]

Negative

The negative result indicate the absence of canine *Dirofilaria immitis* antigen.

Positive

The positive result indicate the presence of canine *Dirofilaria immitis* antigen.

Invalid

The result might be considered invalid. The sample should be retested.

Test results can be read by using test device only, but the results are unreliable. For more accuracy of test results, reading the results using V200 analyzer is recommended.

1. Only control (“C”) line in the result window indicate “Negative”.
2. Test (“T”) line and control (“C”) line within the result window indicate “Positive”.
3. If the control (“C”) line does not appear, the result might be considered invalid. The sample should be retested.

NOTE

The analyzer test result of a sample is given as Positive(+)/Pos(+) or Negative(-)/Neg(-) with a COI (cutoff index) value. COI is calculated that a measured signal is divided by an appropriate cutoff value.

- Test results of a COI ≥ 1.00 are considered positive for canine *Dirofilaria immitis* antigen.
- Test results of a COI < 1.00 are considered negative for canine *Dirofilaria immitis* antigen.

LIMITATIONS OF THE TEST

1. Although the Vcheck CHW Ag Test kit is very accurate in detecting canine *Dirofilaria immitis* antigen, a low incidence of false results can be occurred. Other clinically or laboratory tests might be required if questionable results are obtained. As other diagnostic tests, a definitive clinical diagnosis should not be based on the result of a single test, but should be diagnosed by the veterinarian after all clinical and laboratory findings have been evaluated.
2. Neither the quantitative value nor the rate of canine *Dirofilaria immitis* antigen concentration can be determined by this qualitative test.
3. The reading window may show a light pink background coloration; this will not affect the accuracy of the results.
4. BioNote and its distributors cannot be held responsible for the consequences of misuse or misinterpretation of the results given by the test.

SCREEN MESSAGES AND TROUBLE SHOOTING

[V200]

Error message	Error description
Contaminated Device	The test device is damaged or inserted improperly. Solution: Discard the test device and retest with a new test device and a new specimen.
Insufficient Sample	An insufficient amount of blood has been applied. Solution: Retest with a new test device with enough specimen, ensuring that blood is placed in to the narrow channel in the top edge of the test device.
Expired Device	The test devices are expired. Solution: Retest with a new test device that is not expired.
Temperature Error	The environmental temperature is above or below the operating range of the analyzer. Solution: Move to an area in the acceptable temperature range for the analyzer and perform the test. Do not heat or cool the analyzer artificially.
Printer Connection Fail	The communication between analyzer and barcode or printer has failed. Solution: Reconnect the analyzer and external device.
Barcode Error	If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Extremely Total hemoglobin	The measured total hemoglobin is out of the range of 7 to 23g/dL. Solution: This error occurs when a specimen has a total hemoglobin in the abnormal range. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Result: Invalid	The test is invalid. Solution: Retest with a new test device and a new patient specimen. If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Calibration Overdue	The calibration is overdue. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.
Not Supported Device	Loading a test device that is not supported by the analyzer. Solution: Check whether the test device is manufactured by BioNote, Inc.
EEE	Internal error has occurred. Solution: If the error continues after turning ON/OFF the analyzer, please contact BioNote, Inc.

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